



FEATURE COMPARISON

The Omnipod® System vs. the Omnipod DASH™ System

Already a Podder™, or interested in becoming one? Both the Omnipod® System and the Omnipod DASH™ System offer key benefits for insulin management. See below for what's new with Omnipod DASH™.

 omnipod INSULIN MANAGEMENT SYSTEM	FEATURES	 omnipod DASH INSULIN MANAGEMENT SYSTEM
THE POD		
Yes	Waterproof*	Yes
Yes	72 hours of non-stop insulin delivery†	Yes
Yes	Tubeless	Yes
Radio frequency	Wireless Connectivity	Bluetooth®
Available through DME	Coverage Benefits	No lock-in period
Arm, abdomen, lower back, buttocks, leg, thigh	Placement	Arm, abdomen, lower back, buttocks, leg, thigh
1.53" x 2"	Size	1.53" x 2"
Yes	Small, flexible auto-canula insertion	Yes
THE PERSONAL DIABETES MANAGER (PDM)		
Yes	Manages insulin delivery based on custom settings	Yes
No	Touchscreen	Yes
No	Wi-Fi Enabled	Yes
Integrated with FreeStyle	Blood Glucose Meter (BGM)	Optimized for use with the CONTOUR® NEXT ONE BGM provided with each Podder™ kit
No	iOS Compatibility	<ul style="list-style-type: none"> • Omnipod DISPLAY™ App, for access to insulin data on an iOS smartphone‡ • Omnipod VIEW™ App, for caregivers to remotely monitor up to 12 users from their iOS smartphones‡ • iOS Today View alongside Dexcom® Continuous Glucose Monitor (CGM) data
No	Ability to Set Zero Basal Rates	Yes
1.0g carb/unit	Insulin to Carb Ratio Increments	0.1g carb/unit
Yes, with 1,000 common food items	Food Library	Yes, with a CalorieKing® Integrated Food Library with over 80,000 food items
No	Firmware Updates	Yes
(2) AAA Non-Alkaline	Battery	(1) Rechargeable Lithium Ion (<i>chord included</i>)
<ul style="list-style-type: none"> • Side-by-side and touching the Pod during activation and priming • Within 5 feet of the Pod to deliver boluses, make changes to delivery, and/or update Pod status • Not required to be near the Pod when the Pod is active and delivering basal insulin 	Range to Pod	<ul style="list-style-type: none"> • Side-by-side and touching the Pod during activation and priming • Within 5 to 50 feet of the Pod to deliver boluses, make changes to delivery, and/or update Pod status • Not required to be near the Pod when the Pod is active and delivering basal insulin
Cable upload through software, kiosk, transmitter and web application	Glooko Data Management	Cable upload through software, kiosk, transmitter and web application
Varies depending on coverage	Cost	Included in the starter kit at no additional cost

Insulet Corporation ▪ 100 Nagog Park ▪ Acton, MA 01720 ▪ 800.591.3455 ▪ 978.600.7850 ▪ myomnipod.com

* The Pod has a waterproof IP28 rating for up to 25 feet for 60 minutes. The Personal Diabetes Manager is not waterproof.

† Up to 72 hours of insulin delivery

‡ Insulin dosing decisions should not be made based on data displayed on the Omnipod DISPLAY™ or Omnipod VIEW™ app. Always follow the instruction in the User Guide that came with the PDM.

§ The Dexcom System does not have integrated functionality with the Omnipod DASH™ System. iPhone widgets are mobile application shortcuts that are visible after swiping right from the home screen to the iPhone's Today View.

©2019 Insulet Corporation. Omnipod, the Omnipod logo, Podder, DASH, the DASH logo, Omnipod DISPLAY and Omnipod VIEW are trademarks or registered trademarks of Insulet Corporation. All rights reserved. All other trademarks are the property of their respective owners. The Bluetooth® word mark and logos are registered trademarks owned by the Bluetooth SIG, Inc. and any use of such marks by Insulet Corporation is under license. The use of third party trademarks does not constitute an endorsement or imply a relationship or other affiliation. INS-ODS-12-2019-00154 v1.0